mamajoo®

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EN Dear Consumer,

Thank you for choosing Mamajoo products. Mamajoo products meet the stringent EU standards as they are designed and produced with highest quality BPA free raw materials combined with latest high technology, by considering health and safety. For more information and your questions, you can find our contact details at the end of this instructions leaflet. Please keep this leaflet for future references.

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ATTENTION

1. Please read all instructions carefully before using.
2. Do not clean or sterilize the motor unit and its components, the electrical adapter and the air tubing and their inlets because they are not in contact with breast milk. Before using for the first time, remove all parts of your pump that come in contact with your breast milk, wash them thoroughly with warm soapy water and rinse them thoroughly and sterilize them using Mamajoo sterilizers or by boiling them in water for 5 minutes. Clean all parts after each use and sterilize all parts before each use. To prevent damage to the pump components, do not sterilize longer than recommended.
3. NEVER expose parts that do not come into contact with breast milk, such as the motor unit and its components, electrical adapter, air tubing and inlets to water or moisture, do not clean with a wet cloth.
4. Avoid cleaning with anti-bacterial detergents and ablutents that may cause damage or cracking on the product. In such a case please do not use the damaged pieces.
5. Use Mamajoo Electronic USB Breast Pumps with ONLY the original adapter. Check if it turns off correctly after each use and unplug the power cord. Thus, you save energy and extend the life of the product.
6. Do not exceed the capacity of the breast milk storage / feeding container to avoid motor unit damages. If you get more than 120 ml milk when you pump please use a larger feeding bottle.
7. Keep out of reach of children and babies.
8. Extra hard and fast milk expression may cause pain so that please ensure the power is right for you. We do not recommend you to use your breast pump at the last suction mode except in special cases such as Mastitis.
9. Please check before each use that the comfort cushion fits correctly to the funnel, otherwise milk can leak out.
10. When a problem occurs, unplug the power cord or remove the product from the USB source. In case of any requirements of service, please contact us to direct you to our technical customer service.
11. Please be very careful not to pull too strong the stopper of Mamajoo Electronic USB Double Breast Pump.
12. There should be strict compliance with the instructions. Our company will not be held responsible for any damage caused by improper use.
13. Do not use the breast pump in the bath or in the shower.
14. Do not place or store the breast pump near a tub or sink.
15. Do not reach for a product that has fallen into water. Unplug immediately.
16. Always unplug the breast pump immediately after use.

17. This product is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance
 18. Do not allow children or pets to play with the motor unit, the adapter or accessories, as these parts may pose a choking or strangulation danger.
 19. Inspect the breast pump, including the adapter, for signs of damage before each use. Do not use the breast pump if the adapter or plug is damaged, if it does not work properly, if it has been dropped or submerged in water.
 20. Do not use the breast pump outdoors, where aerosol spray products are used or where oxygen is administered.
 21. The breast pump is not suitable for use in the presence of flammable anaesthetic mixtures with air or with oxygen or nitrous oxide.
 22. There are no user serviceable parts inside the motor unit of the breast pump. Do not attempt to open, service or repair the breast pump yourself. If you do, your guarantee becomes invalid.
 23. Keep the adapter and the silicone tubes away from heated surfaces. Only use the adapter indicated on the appliance.
 24. Do not attempt to remove the pump body from your breast while under vacuum. Switch off the appliance and break the seal between your breast and the pump funnel with your finger. Remove the pump from your breast.
 25. Never use the breast pump while you are pregnant, as pumping can induce labour.
 26. Never use the breast pump while you are sleepy.
 27. Do not expose the breast pump to extreme heat and do not place it in direct sunlight.
 28. Always start milk expression at massage mode to stimulate let-down reflex. Do not continue pumping for more than 5 minutes at a time if you do not succeed in expressing any milk. Try to express at another time during the day. If the process becomes very uncomfortable or painful, stop using the pump and consult your breastfeeding advisor.
 29. Check the product before each use and replace any worn parts. Expressing milk with worn out parts may result motor unit damage due to milk leakage to the motor unit.
 30. If the duck valve is not cleaned well, expressing milk properly will not be possible.
- After each use, remove the duck valve and clean it well to remove any residual milk.

MAINTENANCE

1. Clean the motor unit **ONLY** with a dry cloth. Do not use abrasive or anti-bacterial detergents.
2. Check regularly that the air tube connections are made correctly and the duck valve is not damaged.
3. The pump body and the breast milk storage containers can be washed in the dishwasher top rack. However a color change can occur due to foods.

The air tube cannot be sterilized or washed. The water left in the tube might destroy the motor part.

TECHNICAL FEATURES

- The advanced intelligent microprocessor technology provides complete control.
- Processes and signs can be easily monitored from LED indicators.
- It can be used with an external charger (powerbank) for mobile applications. External charger must have min. 2600 mA real capacity.

Product Code: MMJ2858

Adaptor: Input: 100- 240V ~ 50/60Hz Output: 5-6V 1A DC

Run Time: 0-30 minutes

Weight: 0.170 kg

Product Code: MMJ2841

Adaptor: Input: 100- 240V ~ 50/60Hz Output: 5-6V 1A DC

Run Time: 0-30 minutes

Weight: 0.150 kg

Electronic USB Double Breast Pump product components

- a) Cylinder cover
- b) Silicone diaphragm
- c) Pump body
- d) Comfort cushion
- e) Valve
- f) Bottle teat, dome cap, screw ring and sealing disc
- g) 1 x Breasts milk storage container (150 ml) 1 x Bottle body (250 ml)
- h) Breast milk storage container base
- i) Motor unit
- j) Air tube
- k) Air tube – Motor unit insert
- l) Stopper
- m) AC/DC adapter and cable

Electronic USB Single Breast Pump product components


- a) Cylinder cover
- b) Silicone diaphragm
- c) Pump body
- d) Comfort cushion
- e) Valve
- f) Bottle teat, dome cap, screw ring and sealing disc
- g) Breasts milk storage container (150 ml)
- h) Breast milk storage container base
- i) Motor unit
- j) Air tube
- k) Air tube – Motor unit insert
- l) Stopper
- m) AC/DC adapter and cable

NEVER sterilize these parts: i, j, k, l and m

HOW TO ASSEMBLE

Before starting to assemble, please wash your hands.

Before using wash and sterilize the mechanical pieces of the breast pump.

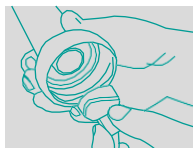
Push the valve to the pump body tightly.

Place the silicone diaphragm onto the pump body.


Place the cylinder cover onto the pump body.

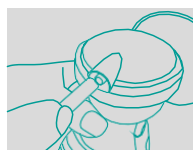
Insert the end of the air tube into the air port of the cylinder cover.


Screw breast milk storage container to the pump body.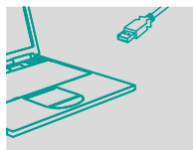

Insert the USB plug into the USB source output for mobile use.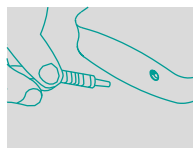


Double Breast Pump
Insert the other end of air tube into the port of the motor unit.

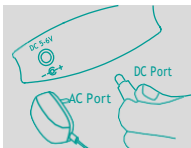
Double Breast Pump
Insert the metal plug at the end of adapter into the adapter input on the motor unit. (5V-6V) Plug in the adapter. (110V-220V)

Double Breast Pump
Make sure the stopper is firmly connected to the second output for single use. In case of an air escape, pump will not function correctly.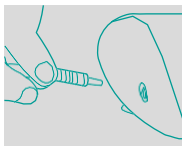

Single Breast Pump
Insert the other end of air tube into the port of the motor unit

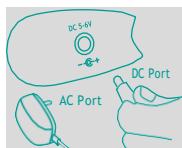Single Breast Pump
Insert the metal plug at the end of adapter into the adapter input on the motor unit (5V-6V) and plug in the adapter (110V-220V).

LED INDICATOR MANUAL

Mamajoo Electronic USB Double Breast Pump

8 different level options of massage mode 12 different level options of suction mode

Double massage mode
Functions on 3th level

Example

Single suction mode
Functions on 7th level

Control Panel

- a) Single Breast massage/suction mode
- b) Double Breast massage / suction mode
- c) On/Off and Double/Single mode changing button
- d) Suction decrease button
- e) Suction increase button
- f) LED level indicator

Mamajoo Electronic USB Single Breast Pump


5 different massage mode options

10 different suction mode options

Control Panel

- a) Massage position
- b) Suction position
- c) On/Off and massage/suction mode changing button
- d) Suction decrease button
- e) Suction increase button
- f) LED level indicator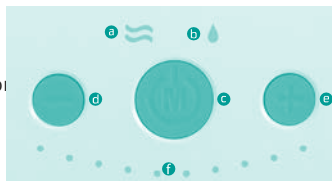

OPERATION

With Adapter:

1. Make sure the breast pump is assembled correctly.
 2. For single use of Mamajoo Double Breast Pump, insert the stopper which is located on the motor unit tightly into the second pump air tube input.
 3. Sit on a comfortable chair and relax, hold the pump with your hand which you feel comfortable with. Do not use breast pump in leaning position.
 4. Center the pump funnel with comfort cushion over your nipple. Make sure it seals your breast completely.
 5. Plug in and press 'On/Off' button to start suction. We recommend you to start with massage mode to stimulate your milk flow.
 6. Mamajoo Breast Pumps mimic baby's natural suckling rhythm.
- 6.1. When you press the On/Off button of Mamajoo Electronic USB Single Breast Pump one time, indicator LED at the upper left lights up for the massage mode, if you press the button for a second time indicator LED at the upper right lights up for the suction mode. The massage mode is for to alert the 'let-down' reflex which is like the baby starts to suck the mother's breast. At the first 5 levels for the massage mode; after each 10 short shots a long shot occurs, when your milk starts to flow press the "On/Off" button again and switch the suction mode. You can select the desired expression mode between 1-10 power levels and with "+" or "-" buttons you can adjust suction power. Mothers with nipple problems and sensitivities can also milk in massage mode. Mamajoo Electronic USB Double/Single Breast Pump saves in memory the last steps you used with its intelligent microprocessor technology. At your next use, the breast pump will start on massage mode and on the last level you used.
- 6.2. Once you press "On/Off" button, Mamajoo Electronic USB Double/Single Breast Pump starts to run on single mode for offering single use or double use options to the mother. Both for single use or double use first 4 levels are for massage mode and the levels from 5 to 10 are for the suction mode. The massage mode is for to alert the 'let-down' reflex which is like the baby starts to suck the mother's breast. At the first 4 levels for massage mode which occurs after each 10 shots short and a long shot when your milk starts to flow press "+" key again and switch the suction mode. You can select the desired expression mode between 5-10 power levels. Mothers with nipple problems and sensitivities can also milk in massage mode. If you would like to use single suction, make sure that the pump is on single mode. Mamajoo Electronic USB Double/Single Breast Pump, saves in memory the last steps you used with its intelligent microprocessor technology. At your next use, the breast pump will start on single suction mode and on the last level you used.

7. When the expression is finished, press "On/Off" button for 2 seconds and unplug it. Take out the breast milk storage container from the pump. Place the Mamajoo sealing disc on the breast milk container and put into the fridge or the freezer or feed your baby directly.
8. Mamajoo Electronic USB Breast Pumps automatically shuts off after it runs 30 minutes with its intelligent micro-processor technology.

External Charger Operation:

In principle, there are not much differences to run the product with an external charger. Instead of plug it with an adapter you can use a fully charged external charger (power-bank, computer etc.) Make sure that the external charger power source is correct. (See Technical Features)

WARNING

The product should be moved by considering the indicated transport direction on the box and by protecting from high temperatures, fire and impacts in its original package during transportation and shipping,

WARRANTY

1. This product is under guarantee of our company for 2 years after date of purchase for the first user. Please contact us or the retailer that you purchased the product from for warranty claims.
2. This warranty does not cover wear or occur from regular use, defects and damages caused by misuse, abuse and improper storage, failure to comply the manual instructions, malfunctions from using improper components.
3. The responsibility of the usage the product belongs to the user.
4. Please keep your receipt as a proof of purchase for any warranty claim.

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mamajoo GmbH