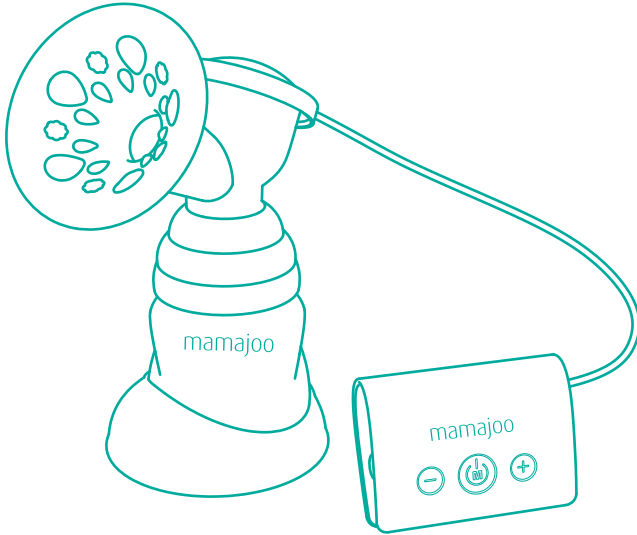
mamajoo®

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Adaptör: Input: 100- 240V ~ 50/60Hz Output: 5-6V 1A DC

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EN Dear Consumer,

Thank you for choosing Mamajoo products and we wish you to raise your baby with health and happiness. Mamajoo products are produced with highest quality BPA free raw materials that meet the stringent EU standards combined with the latest technology, with health of mothers and babies a priority. For more information and questions about our products, you can find our contact details at the end of this instructions leaflet. Please keep this leaflet for future references.

TECHNICAL FEATURES

- The advanced intelligent microprocessor technology provides complete control.
- Processes and signs can be easily monitored from the LED indicator.
- It can be used with an external charger (powerbank) for mobile use. External charger must have min. 2600 mAh real capacity.

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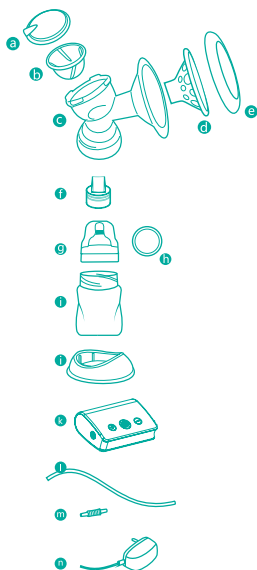
ATTENTION

1. Please read the instruction manual carefully before using this product.
2. Do not clean or sterilize the motor unit and its components, the electrical adapter and the air tubing and their inlets as they are not in contact with breast milk, otherwise the water that passes through may damage the motor unit. Before first use, remove all parts of your pump that come in contact with breast milk, hand wash them thoroughly with warm soapy water, rinse them thoroughly and sterilize them using Mamajoo sterilizers or by boiling them in water for 5 minutes. While boiling, make sure the parts do not make contact with the sides of the pot, as this may result in irreversible damage which you will be responsible for. Clean all parts after each use and sterilize all parts before each use. To prevent damage to the pump components, do not sterilize longer than recommended.
3. NEVER expose parts that do not come into contact with breast milk, such as the motor unit and its components, electrical adapter, air tubing and inlets to water or moisture, do not clean with a wet cloth.
4. Avoid cleaning with anti-bacterial detergents and cleaners that may cause damage or cracking on the product. In such a case please do not use the damaged pieces.
5. Use Mamajoo Electronic USB Breast Pumps with ONLY the original adapter. Check if it turns off correctly after each use and unplug the power cord. This way, you save energy and extend the life of the product.
6. Do not exceed the capacity of the breast milk storage / feeding container to avoid motor unit damage. If you get more than 120 ml milk when you pump please use a larger feeding bottle.
7. Keep out of reach of children and babies.
8. Extra hard and fast milk expression may cause pain so that please ensure the power is right for you. We do not recommend you to use your breast pump at the last suction mode except in special cases such as Mastitis.
9. Please check before each use that the comfort cushion fits correctly to the funnel, otherwise milk can leak out.
10. When a problem occurs, unplug the power cord or remove the product from the USB source. In case of any requirements of service, please contact us to direct you to our technical customer service.
11. Make sure your breast is dry and clean before pumping.
12. There should be strict compliance with the instructions. Our company will not be held responsible for any damage caused by improper use.
13. Do not use the breast pump in the bath or in the shower.
14. Do not place or store the breast pump near a tub or sink.
15. Do not reach for a product that has fallen into water. Unplug immediately.
16. Always unplug the breast pump immediately after use.

- 17.** This product is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance
- 18.** Do not allow children or pets to play with the motor unit, the adapter or accessories, as these parts may pose a choking hazard.
- 19.** Inspect the breast pump, including the adapter, for signs of damage before each use. Do not use the breast pump if the adapter or plug is damaged, if it does not work properly, if it has been dropped or submerged in water.
- 20.** Do not use the breast pump outdoors where aerosol spray products are used or where oxygen is administered.
- 21.** The breast pump is not suitable for use in the presence of flammable anaesthetic mixtures with air or with oxygen or nitrous oxide.
- 22.** There are no user serviceable parts inside the motor unit of the breast pump. Do not attempt to open, service or repair the breast pump yourself. If you do, your guarantee will be voided.
- 23.** Keep the adapter and the silicone tubes away from heated surfaces. Only use the adapter indicated on the appliance.
- 24.** Do not attempt to remove the pump body from your breast while under vacuum. Switch off the appliance and break the seal between your breast and the pump funnel with your finger. Remove the pump from your breast.
- 25.** Never use the breast pump while you are pregnant, as pumping can induce labour.
- 26.** Never use the breast pump while you are sleepy.
- 27.** Do not expose the breast pump to extreme heat and do not place it in direct sunlight.
- 28.** Always start milk expression at massage mode to stimulate the lactation reflex. Do not continue pumping for more than 5 minutes at a time if no milk is being expressed. Try to express at another time during the day. If the process becomes very uncomfortable or painful, stop using the pump and consult your breastfeeding advisor.
- 29.** Check the product before each use and replace any worn parts. Expressing milk with worn out parts may result motor unit damage due to milk leakage to the motor unit. Silicone units such as the air tube, valve and diaphragm should be changed every 4-12 weeks as they become worn depending on usage. Other parts should be changed when damaged.
- 30.** The silicone valve (f) can be clogged if not cleaned properly, making proper milk expression impossible. After each use, remove the duck valve and clean it thoroughly to remove any residual milk.

Product components

- a)** Cylinder Cover
- b)** Silicone Diaphragm
- c)** Pump Body
- d)** Silicone Comfort Cushion
- e)** Hygiene Cover
- f)** Silicone Valve
- g)** Bottle Teat, Cap & Sealing Ring
- h)** Bottle sealing disc
- i)** Breasts milk storage container (160 ml)
- j)** Breast milk storage container base
- k)** Motor unit
- l)** Air tube
- m)** Air tube – Motor unit insert
- n)** AC/DC adapter and cable

NEVER sterilize these parts: k, l, m, n

MAINTENANCE

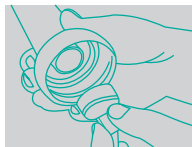
- 1.** Clean the motor unit **ONLY** with a dry cloth. Do not use polishing, abrasive or anti-bacterial detergents.
- 2.** Check regularly that the air tube connections are made correctly and the silicone valve is not damaged.
- 3.** The pump body and the breast milk storage containers can be washed safely in a dishwasher. However a color change can occur due to foods.

The air tube should not be sterilized or washed. The water left in the tube may destroy the motor part.

HOW TO ASSEMBLE

Before starting to assemble, please wash your hands.
Before using wash and sterilize the mechanical pieces of the breast pump.


Push the valve to the pump body tightly.

Place the silicone diaphragm onto the pump body and make sure it fits comfortably.

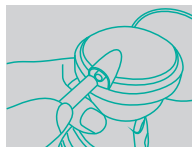
Place the cylinder cover onto the pump body.


Insert the end of the air tube into the air port of the cylinder cover

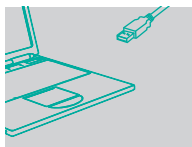
Screw breast milk storage container to the pump body.

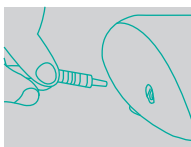Insert the USB plug into the USB source output for mobile use.

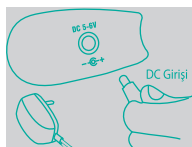Insert the other end of air tube into the port of the motor unit.

Insert the metal plug at the end of adapter into the adapter input on the motor unit (5V-6V) and plug in the adapter (110V-220V).

LED INDICATOR MANUAL

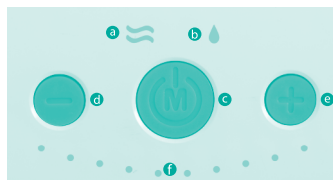
5 different message mode options

10 different suction mode options

Control Panel

- a) Massage position
- b) Suction position
- c) On/Off and massage/suction mode switching button
- d) Suction decrease button
- e) Suction increase button
- f) LED level indicator

OPERATION

With Adapter:

1. Make sure the breast pump is assembled correctly.
 2. Sit on a comfortable chair and relax, hold the pump with your hand which you feel comfortable with. Do not use breast pump in leaning position.
 3. Center the pump funnel with the comfort cushion over your nipple. Make sure it seals your breast completely.
 4. Plug in and press 'On/Off' button to start suction. We recommend you to start with massage mode to stimulate your milk flow.
 5. Mamajoo Breast Pumps mimic baby's natural suckling rhythm.
- 5.1** When you press the On/Off button of Mamajoo Electronic USB Single Breast Pump one time, indicator LED at the upper left lights up for the massage mode, if you press the button for a second time indicator LED at the upper right lights up for the suction mode. The massage mode is to alert the 'let-down' reflex which is similar to the baby starting to suck the mother's breast. At the first 5 levels for the massage mode; after every 10 short pulls a long pull occurs, when your milk starts to flow press the "On/Off " button again and switch the suction mode. You can select the desired expression mode between 1-10 power levels and with "+" or "-" buttons you can adjust suction power. Mothers with nipple problems and sensitivities can also express milk in massage mode. Mamajoo Electronic USB Single Breast Pump saves in memory the last steps you used with its intelligent microprocessor technology. At your next use, the breast pump will start on massage mode and on the last level you used.
6. When the expression is finished, press the "On/Off" button for 2 seconds and unplug it. Take out the breast milk storage container from the pump. Place the Mamajoo sealing disc on the breast milk container and put it into the fridge or the freezer or feed your baby directly.
 7. Mamajoo Electronic USB Single Breast Pump automatically shuts off after running for 30 minutes with its intelligent microprocessor technology.

With an External Charger:

In principle, there are not many differences when operating the product with an external charger. Instead of plugging it in with an adapter you can use a fully charged external charger (power- bank, computer etc.) Make sure that the external charger power source is correct. (See Technical Features)

WARRANTY

1. This product is under guarantee of our company for 2 years after date of purchase for the first user. Please contact us or the retailer that you purchased the product from for warranty claims.
2. This warranty does not cover wear occurring from regular use, defects and damages caused by misuse, abuse and improper storage, failure to comply with the manual instructions and malfunctions from using improper components.
3. The responsibility of the usage the product belongs to the user.
4. Please keep your receipt as a proof of purchase for any warranty claims.

FOR SERVICE POINTS
SERVİS NOKTALARI İÇİN

Üretici / Manufacturer

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Mimar Sinan Mah. Özgürlük Cad. No:77/2
Sultanbeyli-İSTANBUL Tel: 0216 573 62 00 (pbx)

E-mail:info@mamajoo.com

mamajoo GmbH

E-mail:info@mamajoo.de

www.mamajoo.com

mamajoo