mamajoo®

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EN KYBELE Wearable Double Breast Pump

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USER INSTRUCTIONS

Dear Consumer,

Thank you for choosing Mamajoo products. We are very happy to be your side during your motherhood journey.

INSTRUCTIONS & CAUTIONS!

Please read the instructions carefully before using the product and keep for future references.

1. Never expose to water or moisture the parts like motor unit, the power cable, air tube and adapter (f, j, h) that do not come into contact with breast milk as this may damage them, clean only with a dry cloth.
2. Before first use, remove the pump parts that come into contact with breast milk, hand wash with warm soapy water and rinse thoroughly. Sterilize in Mamajoo sterilizers or by boiling in a large pot of water for 5 minutes. Ensure that the milk container or other parts do not touch the edges of the pot to avoid any damage. Do not sterilize for longer than recommended time, otherwise the warranty will be void.
3. Clean the product after each use and assemble according to the assembly guide. Make sure that no small parts are missing, otherwise the product will not work.
4. To avoid damaging the motor, do not tilt the milk container and do not exceed its capacity.
5. Check the product before each use. Silicone parts like valve and diaphragm should be replaced between 4-12 weeks as they wear out by usage, other parts should be replaced immediately when damaged.
6. If the silicone valve is not cleaned properly, it becomes clogged and you can not express milk correctly. After each use, remove the valve and clean the milk residue thoroughly.
7. If the silicone comfort seal does not fit properly in the milk container, milk will leak out, so check it at each use.

8. Make sure your hands and pump parts are clean and your breasts are dry before expressing milk. Always start the pump in massage mode first to stimulate your milk let-down, then switch to expression mode. If you can not express milk, do not pump more than 5 minutes at a time, try again later.
9. Make sure the suction level is suitable for you, as using a rhythm too powerful or fast can cause pain. We recommend you not to use the highest level of your breast pump, except special cases such as mastitis.
10. Do not try to detach the pump from your breast while expressing. Turn off the pump, place your finger in between your breast and the silicone comfort seal.
11. Never use the breast pump while you are pregnant or asleep.
12. Do not leave your breast pump in extremely hot environments, near heat sources and under direct sunlight.
13. Keep out of the reach of babies and small children.
14. Never use your breast pump while bathing or in shower, do not keep near the bathtub or sink. NEVER attempt to retrieve or use a product that has fallen into water.
15. Do not use antibacterial detergents and cleaners as they may crack and damage the product. If this happens, do not use these parts.
16. Please follow the instructions and warnings at all times. Our company can not be held responsible for inappropriate usage.
17. There are no user replaceable parts inside the motor unit of the breast pump. If a problem occurs, do not attempt to open or repair the motor unit. Doing so will void your warranty. Please contact us for technical service if necessary.

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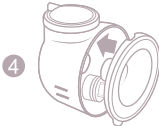
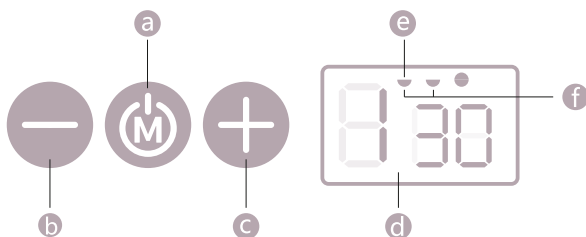
For single use, make sure that the
racket plug is firmly inserted into the
second air tube inlet. The pump will not
vacuum if this inlet is open.

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2.To Start Working & Expression:

- 1.Make sure that all parts are assembled correctly. Place the silicone comfort seal snugly to your breast, making sure there is no gap in between. Keep the pump upright and do not change your position.
- 2.Plug in the product and turn it on by pressing the On/Off (M) button, the LED display will light up in blue.
- 3.You can switch between single and double use by pressing again the On/Off button.
- 4.We recommend to start using with the massage mode first. The first 3 levels are for massage mode, levels between 4-9 are for expression mode.
- 5.Massage mode triggers the “let-down” reflex which stimulates the breast milk production. In the first 3 massage levels, the pump does 10 short and 1 long suction. Mothers with sensitive nipples can express milk in massage mode. When the milk starts to flow, you can switch to level 4 for expression mode and choose one of the levels you prefer.
- 6.When the expression process is over, turn off the pump by pressing the On/Off button for 2 seconds and then unplug the product.
- 7.Gently remove the milk container from your breast. You can transfer your breast milk to Mamajoo Feeding Bottles to feed your baby directly or to Mamajoo Breastmilk Storage Containers to store in freezer for future use.
- 8.For single use, plug in the racket plug into the second air tube inlet on the motor unit and follow above instructions.
- 9.The intelligent memory feature saves your last settings. Next time, your breast pump will start working with the last mode and level used.
- 10.The LED screen counts down from 30 minutes and automatically turns off after.

Using with External Charge:

When you want to operate your pump with an external charge, you can plug the USB connection of the adapter cable to an external USB source (powerbank, car port, computer, etc.) making sure that it is fully charged. The power of the external charging source must have at least 5.000 mA real value.

The air tube should not be sterilized or washed. Any water residue left in the air tube can damage the motor.

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